

DETRO PAA 1500 – INSTRUMENT&ENDOSCOPE HIGH LEVEL DISINFECTANT

DESCRIPTION OF PRODUCT:

%0,15 peracetic acid based high level instrument and endoscope disinfectant. Contains base and activator, after mixing both it generates peracetic acid.

SPECIFICATION OF PRODUCTS:

Chemical-Physical Characteristic

Parameters	M.U	Specifications	Method of Analysis
Physical form	-	Liquid	DETC01
Appearance	-	Liquid	DETC03
Colour	-	Clear	DETC05
Odour	-	Characteristic	DETC07
pH	pH unit, 20°C	6,0 - 8,0	DETC09
Density (20°C)	g/cm ³ , 20°C	1,01 ± 0,05 g/cm ³	DETC11
Solubility in water	-	Soluble	DETC13
Peracetic Acid	W/W	0,15% ± 15	DETC15

*Confirmed by Detrox Quality Control Department.

Microbiological Characteristics

Parameter	M.U.	Specifications	Methods of Analysis
Escherichia coli NCTC 10538	cfu/ml	5 min.	EN 13727
Pseudomonas aeruginosa ATCC 15442	cfu/ml	5 min.	EN 13727
Mycobacterium avium ATCC 15769	cfu/ml	5 min.	EN 14348
Mycobacterium terrae ATCC 15755(Strain, W45)	cfu/ml	5 min.	EN 14348
Candida albicans ATCC 10231	cfu/ml	5 min.	EN 13624
Aspergillus niger ATCC16404	cfu/ml	5 min.	EN 13624
Adenovirus	cfu/ml	5 min.	EN 14476
Poliovirus	cfu/ml	5 min.	EN 14476
Norovirus	cfu/ml	5 min.	EN 14476
Bacillus subtilis ATCC 6633	cfu/ml	5 min.	EN 13704

DETRO PAA 1500 – INSTRUMENT&ENDOSCOPE HIGH LEVEL DISINFECTANT

INTENDED USE OF PRODUCT:

Usage Area

- Used for disinfection and cold sterilization of general surgical equipments, endoscopes and anesthesia equipments.

Ingredients

- % 0,23 peracetic acid, auxiliary substances

Usage form

- Detro PAA 1500 is used at manually or at endoscope washer machines for endoscopes disinfection.
- Before usage, pour activator into based drum, mix base and activator by shaking 10 min. to generate activated solution.
- At first usage, completely clean instruments, basins, endoscopes with enzymatic cleaner to prevent chemical reactions at instruments and endoscopes.
- Immerse instruments into activated solution and wait effect time (5 minutes)
- After effect time, take out instruments and rinse with sterile water.
- Control activity of solution by test strips.
- Solution can be used during 7 days. Activity should be controlled by test strips.
- Do not use on aluminium, nickel, chrome covered instruments.

Microbiological effect

Bactericidal, fungicidal, virucidal, tuberculoidal, sporicidal in 5 min.

First aid Informations

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL
General Manager

DETRO HEALTHCARE KİMYA SAN. A.Ş.
Muhür Mersinli Ağız, Ankara, Çankaya
No:7 Etiler, Beşiktaş, İstanbul T.C.
Tic. Sic. No: 271144 - Mers. No: 08100000000000000000