

DETRO INSTRUCLEAN – INSTRUMENT&ENDOSCOPE CLEANER

DESCRIPTION OF PRODUCT:

Cleaning solution for precleaning of instruments and endoscopes before disinfection process.

SPECIFICATION OF PRODUCTS:

Chemical-Physical Characteristic

Parameters	M.U	Specifications	Method of Analysis
Physical form	-	Liquid	DETC01
Appearance	-	Limpid	DETC03
Colour	-	Colourless	DETC05
Odour	-	Characteristic	DETC07
pH	pH unit, 20°C	10-12	DETC09
Density (20°C)	g/cm ³ , 20°C	1,05 ± 0,1 g/cm ³	DETC11
Solubility in water	-	Soluble	DETC13

*Confirmed by Detrox Quality Control Department.

DETRO INSTRUCLEAN – INSTRUMENT&ENDOSCOPE CLEANER

INTENDED USE OF PRODUCT:

Product Properties

- Effects 5-10 min at %0,5 concentration; At ultrasonic machines effects 2 min.
- Remove contaminations
- Has corrosion inhibitors

Usage Areas

- It is used for cleaning of general surgical equipments, endoscopes and anesthesia equipments etc.

Usage form

- Add 5 ml product into 1 lt water.
- Immerse instruments during 5-10 min. If possible, wash a little bit instruments before immersing.
- If there is wizen contaminations, prolong waiting time.
- It is not necessary to brush instruments.

Effect time : 5-10 min.

Note : Increasing of water temperature accelerate claning due to it also increases activation. Solution should renew daily.

Preparation of Product Concentration

WATER AMOUNT	%0,5 SOLUTION	EFFECT TIME
1 L	5 ml	5 min.
5 L	25 ml	5 min.
10 L	50 ml	5 min.
15 L	75 ml	5 min.
20 L	100 ml	5 min.

Ingredients

Nonionic tencides, corrosion inhibitors, complexing agents

First aid informations:

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL
General Manager

DETRO HEALTHCARE KİMYA SAN. A.Ş.
Atatürk Mahallesi Akşirli Yolu-Çekirge Caddeşi
No: 7 Etiler - Beşiktaş / İstanbul / Türkiye
Tel: 0 212 659 77 62 - 0 212 659 77 63
Fax: 0 212 659 77 62 - 0 212 659 77 63

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📠 212 659 77 63

✉ info@detrox.com.tr - www.detrox.com.tr